



SureStep™

Drug Screen Tests



Be confident... Be definite... Be sure



SureStep™

Drug Screen Tests

are simple, one step tests for the detection of drugs of abuse in urine.

SureStep™ Drug Screen Tests feature both single and multiple panel tests providing additional flexibility for targeted testing or the simultaneous detection of up to 9 drugs with a single urine sample.



SureStep™ Drug Screen Tests provide rapid, accurate and reliable results in just 5 minutes

Features and benefits

- Just one step
- Inbuilt procedural control
- Rapid
- Room temperature storage
- Multiple drugs measured
- Minimal training required
- Confirmation of correctly performed test
- Results in 5 minutes
- Ready to use, no need to refrigerate
- Simultaneous detection of several drugs

www.suresteptest.com



SureStep™ Drug Screen Tests

According to the World Health Organisation 200,000 people die every year from drug abuse. At least 15.5 million people have their quality of life affected by problems associated with drug abuse.¹

Early detection of drug abuse can allow intervention to help reduce the adverse effects and allow appropriate treatment. Simple and accurate tests are therefore essential for use at point of care and in rehabilitation clinics, as well as forming a vital part of workplace assessments, Occupational Health screening and Criminal Justice systems.

The following drugs can be detected with



SureStep™ Drug Screen Tests




- Amphetamines
- Cocaine
- Morphine
- Barbiturates
- Ecstasy
- Opiates
- Benzodiazepines
- Methadone
- Propoxyphene
- Cannabis
- Metamphetamines
- Tricyclic antidepressants

1. World Health Organisation website: www.who.int/substance_abuse/facts/psychoactives/en/index.html

Test Procedure

- 1 Using the transfer pipette, apply 3 full drops of urine (approx. 100 μ L) to each specimen well of the device.
- 2 The test must be read 5 minutes after applying the sample. A pink/red line must appear in the control line region (C) to indicate that the test has worked correctly. Do not interpret results after 10 minutes.

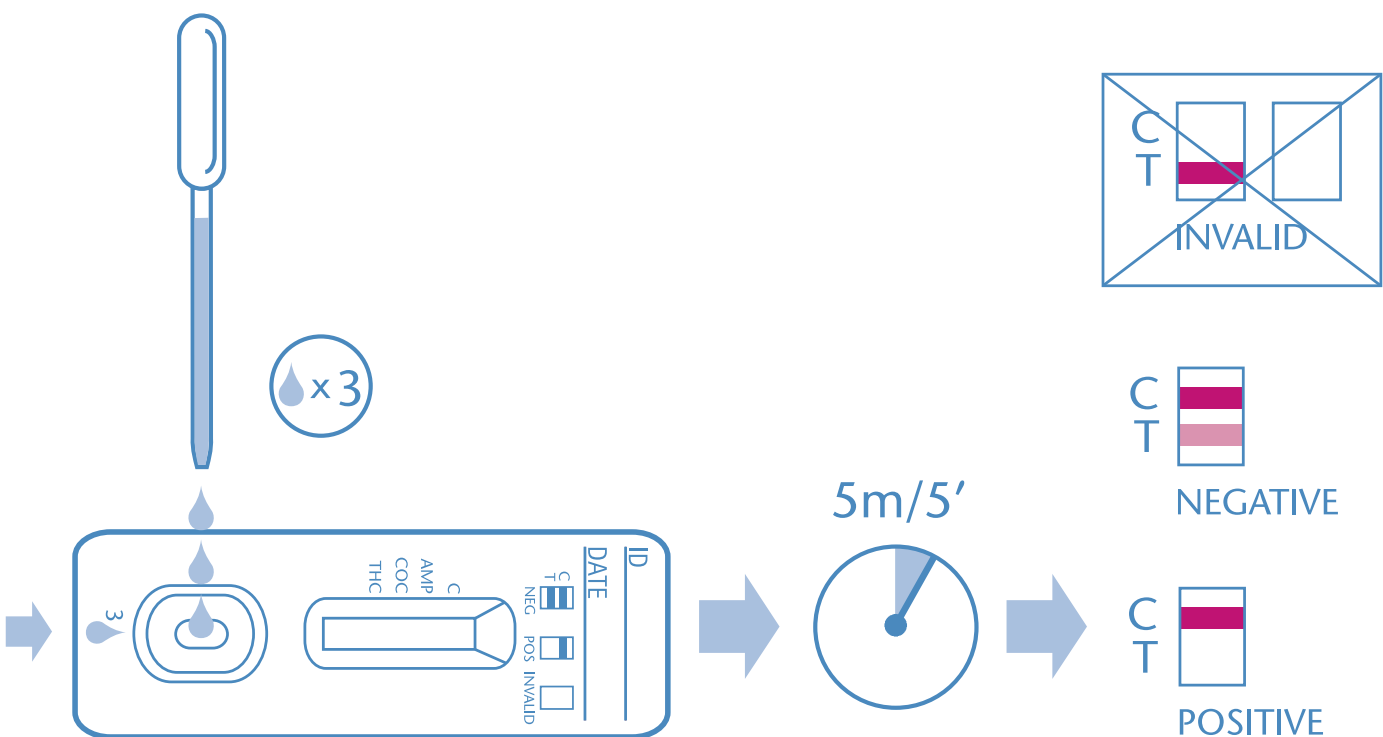



SureStep™

Product Codes & Descriptions

SINGLE ANALYTE TESTS

Product code	Test Description	Cut-off (ng/mL)
DTH-102 (40T)	Surestep THC - cannabis	50
DBZ-U102 (40T)	Surestep Benzodiazepines (BZO 200)	200
DMO-102 (40T)	Surestep Morphine (MOP)	300
DOP-102 (40T)	Surestep Opiates (OPI)	2000
DCO-102 (40T)	Surestep Cocaine (COC)	300
DCO-U102 (40T)	Surestep Cocaine (COC II)	150
DAM-102 (40T)	Surestep Amphetamines (AMP)	1000
DAM-A102 (40T)	Surestep Amphetamines II (AMP II)	300
DME-102 (40T)	Surestep Methamphetamines (MAMP)	1000
DME-U102 (40T)	Surestep Methamphetamines II (MAMP II)	500
DBA-102 (40T)	Surestep Barbiturates (BAR)	300
DMD-102 (40T)	Surestep Ecstasy (MDMA)	500
DTC-102 (40T)	Surestep Tricyclic antidepressants (TCA)	1000
DMT-102 (40T)	Surestep Methadone (MTD)	300
DPP-102 (40T)	Surestep Propoxyphene (PPX)	300
DBZ-102 (40T)	Surestep Benzodiazepines (BZO 300)	300

MULTIPLE PANEL TESTS

Product code	Test Description	
DOA-125 (25T)	Surestep THC/COC	2 panel test
DOA-135 (25T)	Surestep THC/COC/MAMPII	3 panel test
DOA-155A (25T)	Surestep AMP/COC/MAMP/OPI/THC	5 panel test
DOA-155 (25T)	Surestep MOP/AMPII/THC/COC/BZO200	5 panel test
DOA-165B (25T)	Surestep AMPII/BZO300/COC/MDMA/MOP/THC	6 panel test
DOA-165 (25T)	AMPII/BZO300/COC/MAMPII/MOP/THC	6 panel test
DOA-165A (25T)	THC/COC/MOP/AMPII/MAMPII/BZO200	6 panel test
DOA-185 (25T)	AMPII/BAR/BZO300/COC/MAMP/MTD/MOP/THC	8 panel test
DOA-195 (25T)	COC/MAMPII/MOP/THC/AMPII/BAR/BZO300/MTD/TCA	9 panel test

Product Information

Kit size:

Single Analyte Test: 40 Tests
Multiple Panel Test: 25 Tests

Shelf life:

24 months

Storage:

2 - 30°C

Sample type:

Urine

Time to result:

5 minutes

Further Products



SureStep hCG FHC-102



SureStrip hCG FHC-101



SureStep LH FLH-102

Distributed by

enquiry@suresteptest.com

Unipath Limited, Bedford MK44 3UP, United Kingdom
Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

 **inverness medical**
professional diagnostics