



SureStep™



LH Ovulation Test

Be confident... Be definite... Be sure



SureStep™ LH

is a simple test for the detection of LH in urine to aid the identification of the LH surge and predict ovulation. This information can be used to time intercourse and help achieve conception.



SureStep™ LH identifies the LH surge in just 5 minutes

Features and benefits

- | | |
|------------------------------|--|
| • Rapid | Results in 5 minutes |
| • Proven Accuracy of 99% | Reliable |
| • Inbuilt procedural control | Confirmation of correctly performed test |
| • Just one step | Minimal training required |
| • Room Temperature Storage | Ready to use. No need to refrigerate |
| • Long shelf life | Helpful for small-volume users |

www.suresteptest.com




SureStep™

LH Ovulation Test

Test Procedure

- 1 Using the transfer pipette, apply 3 full drops of urine to the specimen well.
- 2 The test must be read 5 minutes after applying the sample. A pink/red line must appear in the control line region (C) to show that the test has worked correctly. Any result that appears after 10 minutes should be ignored. A positive result indicates an LH surge and that ovulation is likely to occur within 24-36 hours.

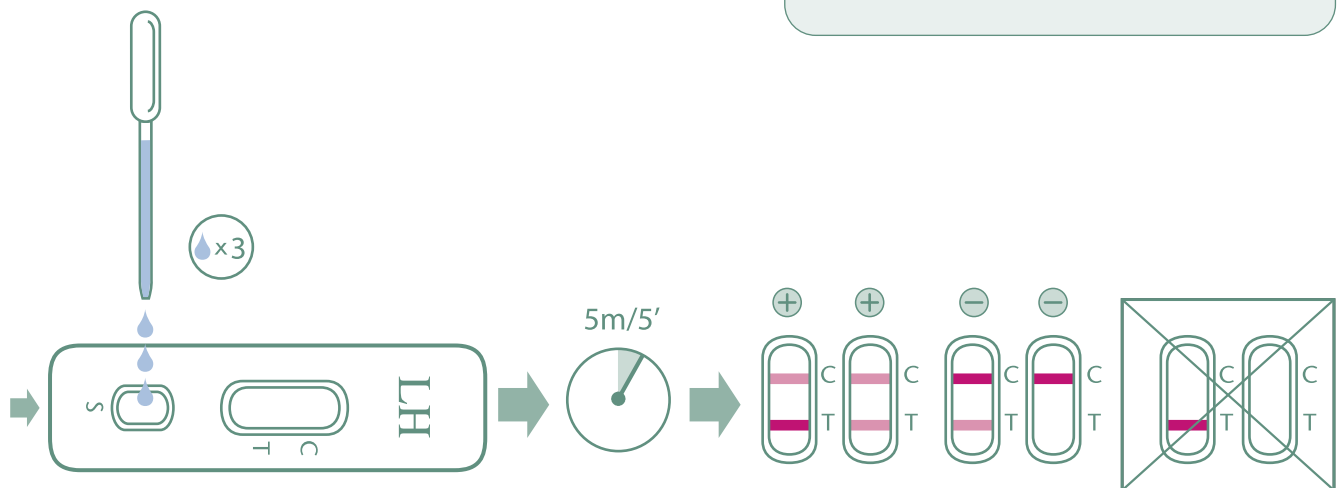




Distributed by

Product Information

Cassette cat no.	FLH-102
Kit size:	40
Shelf life:	24 months
Storage:	2 - 30°C
Sample type:	Urine
Time to result:	5 minutes
Sensitivity:	40 mIU/ml
Specificity:	100%
Accuracy:	>99%

Further Products

-  SureStep hCG FHC-102
-  SureStrip hCG FHC-101
-  SureStep Drug Screen Test (DST)

enquiry@surestep-test.com

Unipath Limited, Bedford MK44 3UP, United Kingdom
Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

 **inverness medical**
professional diagnostics