



SureStrip™

One-Step hCG Pregnancy Test

Be confident... Be definite... Be sure



SureStrip™ hCG

is a simple test for the detection of hCG in urine as an aid to early confirmation of pregnancy



SureStrip™ hCG provides a clear positive or negative result in just 3 minutes

Features and benefits

- Rapid
- Sensitive to 25mIU/ml
- Just one step
- Proven accuracy of >99%
- Inbuilt procedural control
- Room temperature storage
- Long shelf life
- Results in 3 minutes
- Detects pregnancy on day of missed period
- Minimal training required
- Reliable
- Confirmation of correctly performed test
- Ready to use. No need to refrigerate
- Helpful for small-volume users

www.suresteptest.com




SureStrip™

One-Step hCG Pregnancy Test

Test Procedure

- 1 Immerse the test strip vertically in the urine specimen, up to the MAX line, for at least 10-15 seconds.
- 2 Place the test strip on a non-absorbent flat surface. Read after 3 minutes. A pink/red line must appear in the control line region (C) to show that the test has worked correctly. Any result that appears after 10 minutes should be ignored.

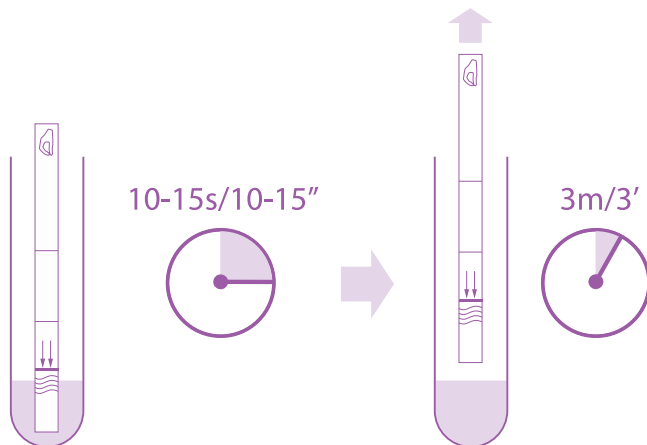

Distributed by


enquiry@surestep.com

Unipath Limited, Bedford MK44 3UP, United Kingdom




Tel: +44 (0)1234 835000 Fax: +44 (0)1234 835009

Product Information
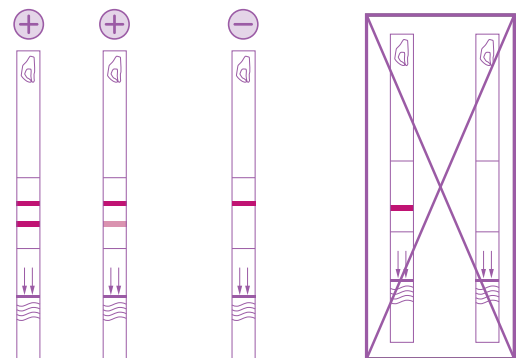
| | |
|------------------|-----------|
| Cassette cat no. | FHC-101 |
| Kit size: | 50 |
| Shelf life: | 24 months |
| Storage: | 2 - 30°C |
| Sample type: | Urine |
| Time to result: | 3 minutes |
| Sensitivity: | 25 mIU/ml |
| Specificity: | 100% |
| Accuracy: | >99% |



Further Products

-  SureStep hCG FHC-102
-  SureStep LH FLH-102
-  SureStep Drug Screen Test (DST)

 **inverness medical**
professional diagnostics